

Pieces and Parts

tConsult Client Software



- Volume Dial
- Bell/Diaphragm Switch
- Chest Piece Plug & Unplug Here
- Chest Piece
- Headphone Hook
- Headphones

Bell/Diaphragm Switch:



- Flip switch up to D for diaphragm settings
- Diaphragm setting can be used to listen to lung sounds, bowel sounds, and some abnormal heart sounds
- Flip switch down to B for bell settings
- Bell setting is used to listen to most heart sounds

Adjustable:

- Volume Dial (clockwise to increase)
- Bell & Diaphragm Switch can be adjusted independently

Dropdown Menus

- Exam Description
- Location of Chest Piece
- Patient Position
- Technique
- Other description
- Interval

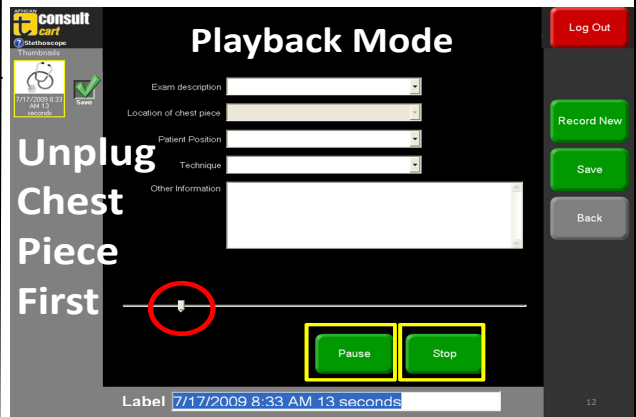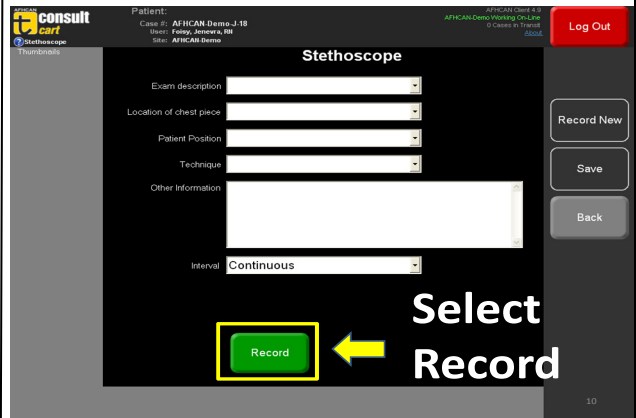

Listening to Clips Using tConsult Web Software



1. Attach USB stethoscope unit to your computer; stethoscope can be used on any PC or laptop with network access

2. You may need to install AFHCAN Web Tools from log in screen; click  button

3. Log in to tConsult Web

4. Open case and click on stethoscope attachment

5. Plug in the headphones

6. Click open & click play

